

# FTS

fentanyl test strip instructions

\* for all tests \*

1



prep shot & put  
needle aside

2



leave residue in  
cooker or baggie



**only Meth & MDMA**



**3**



drop cooker & residue  
into **1 to 2 oz of water**

**4**



hold test strip in cup  
for **15 seconds**



everything but  
Meth & MDMA



3



add a  $\frac{1}{4}$  inch  
of water

4



do not go past  
the blue line

insert strip for  
**15 seconds**

\* for all tests \*

5

take out,  
lay flat,



& wait **2 minutes**

6

**1 line**

appear if fentanyl is found.

**2 lines**

appears if it is not found.



**Use caution.**

Fentanyl may still be present.

**If invalid,**

test with a new strip.

6



1 line = **fentanyl found**



2 lines = **fentanyl not found**



no line or missing line "c" = **unclear/invalid**